

Pride Mobility Products
Australia Pty. Ltd.

PMW6/PMW8
Owner's Manual

www.pridemobility.com.au

Pride Mobility Products
Australia P/L

Walkers

PMW6
PMW8

Manufactured by Maxhealth
15F-6, NO.81, SEC. 1, HSIIN TAI WU ROAD,
HSI CHIH, NEW TAIPEI CITY, TAIWAN, R.O.C.

Imported by Pride Mobility Products Australia Pty. Ltd.
20-24 Apollo Drive, Hallam, Victoria, Australia 3803
PH: 03 8770 9600
FAX: 03 9703 2960

www.pridemobility.com.au

assembly instructions

Do not attempt to adjust or operate your Pride Walker without carefully reading all instructions in this manual.

1. First remove all contents from carton: backrest, handle with tube, walker frame and basket.
2. Open frame and slide padded backrest into frame brackets making sure snap bottoms protrude through holes.
3. Unscrew tightening knob from handle and insert handle tube into frame at desired height lining up holes appropriately. Slide bolts through holes from the inside and screw on tightening knob until secure.





safety precautions

1. Please make sure the walker is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test walker for stability before applying full weight to hand grips.
3. Test the hand brake works before operating the walker. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock brakes before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tyres when significantly worn or damaged.
6. Do not have anyone push you while you are seated on the walker. This is a walking aid only and is not to be used as a transportation device.

hand brake operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop — Pull up slowly on the loop lock hand brake levers with four fingers until unit slows down or comes to a temporary stop.
2. To Lock Wheels in Position — Press down on both loop lock hand brake levers with the palms of your hands until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks — Push up on both hand brake levers with palms of hand to original position

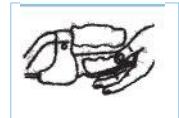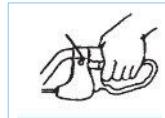

warranty

Your Pride Mobility walker is warranted to be free of defects in materials and workmanship for 12 months of the original consumer purchaser.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Pride device or this warranty, please contact your authorised Pride dealer

“Our goods come with guarantees that cannot be excluded under the Australian Consumer Laws. You are entitled to a replacement or refund for a major failure and for compensation for any foreseeable loss or damage. You are also entitled to have the goods repaired if the goods fail to be of acceptable quality and the failure does not amount to a major failure.”

Pride[®]
*Mobility Products
Australia Pty. Ltd.*

20-24 Apollo Drive, Hallam
Victoria, Australia 3803
PH: 03 8770 9600
FAX: 03 9703 2720

www.pridemobility.com.au